

BIOMET 3i™ Scan Caps To Design BellaTek® Abutments


A Simple Way To Provide Your Customers With Aesthetic, Precise BellaTek® Abutments.



- New scanning device allows for accurate scans on fixture-level impression models
- Scan caps for designing BellaTek Abutments using your 3Shape Scanner
- All BellaTek Certain® Abutments can be created from just two scan caps
- Electronic order submission allows for quicker turnaround



STEP 1: Pour the model.

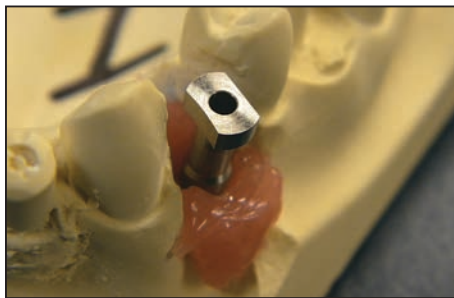

STEP 2: Attach the scan cap to the analog.



STEP 3: Spray the model with an appropriate anti-reflective spray.

STEP 4: Scan the model with your 3Shape Scanner.

STEP 5: Electronically submit model to the BellaTek Production Facility.

STEP 6: Receive an aesthetic, precise BellaTek Abutment.

BIOMET 3i™

BIOMET 3i
4555 Riverside Drive
Palm Beach Gardens, FL 33410
1-888-800-8045
Outside The U.S.: +1-561-776-6700
Fax: +1-561-776-1272
www.biomet3i.com

EC REP
BIOMET 3i
Europe, Middle East & Africa
WTC Almeda Park, Ed. 1, Planta 1ª
Pl. de la Pau, s/n
08940, Cornellà de Llobregat
(Barcelona) Spain
Phone: +34-93-470-55-00
Fax: +34-93-371-78-49



Join Us



Follow Us



Watch Us



Download It

