

CopiOs Extend[®] Membrane

Synergy Of Space And Time

Space Maintaining Conformance

- Designed to conform to the defect with enough structural integrity for space maintenance

Extended Resorption Profile

- Lasts 6 to 9 months,¹ designed to allow implant placement while providing ample time for regeneration

Performs Even When Primary Closure Is Not Attained


- No signs of inflammation or infection in cases where primary closure has not been achieved^{2,3}



Long Lasting, Resorbable Collagen Membrane

Performs When Primary Closure Has Not Been Achieved

Clinical experience with CopiOs Extend Membrane has shown that the membrane shows no signs of inflammation or infection in cases where primary closure has not been achieved. The exposed area is healed by soft tissue covering the exposure within a few weeks while the membrane maintains its barrier function^{1,2}

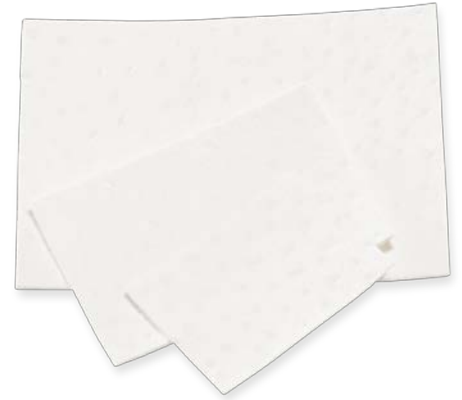

Indications

CopiOs Extend Membrane is intended for use in oral surgical procedures as a resorbable material for:

- Use in augmentation around implants placed in immediate and delayed extraction socket
- Localized ridge augmentation for later implantation
- Alveolar ridge reconstruction for prosthetic treatment
- Filling of bone defects
- Guided bone regeneration in dehiscence defects
- Guided tissue regeneration procedures in periodontal defects

Take A Closer Look



Figure A Clinical image showing the extensive bone loss on the palatal area extending past apex.



Figure B CopiOs Extend Membrane was laid over the graft and tacked under the flap.



Figure C Flap sutured over the membrane.



Figure D Two months postoperative. Excellent healing.

Clinical photos ©2013 Dr. El Chaar†
All rights reserved. Individual results may vary.

Ordering Information

| Catalog Number | Description |
|----------------|------------------------------------|
| 0190Z | CopiOs Extend Membrane, 15 x 20 mm |
| 0191Z | CopiOs Extend Membrane, 20 x 30 mm |
| 0192Z | CopiOs Extend Membrane, 30 x 40 mm |

1 Data on file with Collagen Matrix, Inc.

2 Clinical cases on file with Zimmer Dental, Inc.

3 Clinical cases on file with Collagen Matrix, Inc.

† Dr. El Chaar has a financial relationship with Zimmer Biomet Dental resulting from speaking engagements, consulting engagements and other retained services.

Contact us at 1-800-342-5454 or visit zimmerbiometdental.com

Zimmer Biomet Dental
Global Headquarters
4555 Riverside Drive
Palm Beach Gardens, FL 33410
Tel: +1-561-776-6700
Fax: +1-561-776-1272



Collagen Matrix, Inc.
15 Thornton Rd,
Oakland, NJ 07436



Zimmer Dental SAS
2, place Gustave Eiffel
94528 Rungis Cedex
France

All trademarks are the property of Zimmer Biomet or its affiliates, unless otherwise indicated. All references contained herein to Zimmer Biomet Dental refer to the Zimmer Biomet Dental Division. Due to regulatory requirements, Zimmer Biomet's dental division will continue to manufacture products under Zimmer Dental Inc. and Biomet 3i, LLC respectively until further notice. CopiOs Extend Membrane is manufactured by Collagen Matrix, Inc., and distributed by Biomet 3i, LLC. Please note that not all products and regenerative materials are registered or available in every country/region. Please check with a Zimmer Biomet Dental representative for availability and additional information. ZB0021 REV A 10/16 ©2016 Zimmer Biomet, All rights reserved.

