


# Scan Caps for Certain<sup>®</sup> BellaTek Abutments

A simple way to provide your customers  
with aesthetic, precise BellaTek Abutments

- Scanning device allows for accurate scans on fixture-level impression models
- Scan caps for designing BellaTek Abutments using the CAD/CAM system in your laboratory
- All BellaTek Certain Abutments can be created from just two scan caps
- Electronic order submission allows for quicker turnaround



*BellaTek*<sup>®</sup>  
Abutment

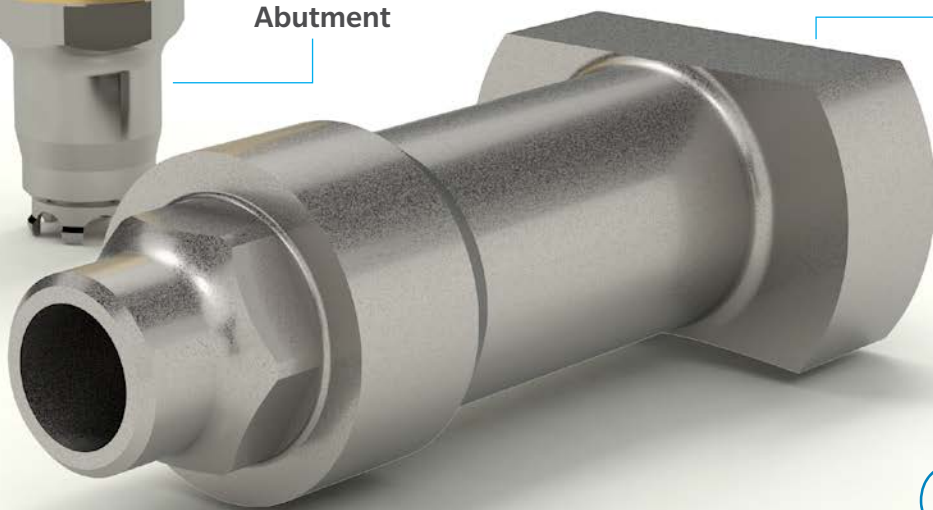


Scan Cap  
For Lab Use Only.



ZIMMER BIOMET  
Your progress. Our promise.®

**Step 1**

Pour the model

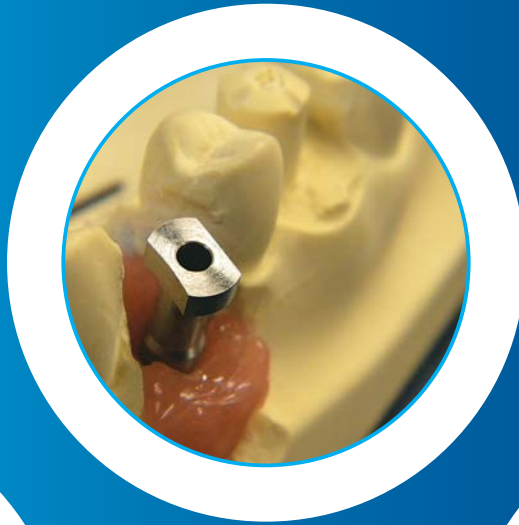

**Step 2**

Attach the scan cap to the analog

**Step 3**

Spray the model with an appropriate anti-reflective spray



**Step 4**

Scan the model with the CAD/CAM system of your choice

**Step 5**

Electronically submit model to the BellaTek Production Facility

**Step 6**

Receive an aesthetic, precise BellaTek Abutment

Contact us at 1-800-342-5454 or visit [zimmerbiometdental.com](http://zimmerbiometdental.com)

Zimmer Biomet Dental  
Global Headquarters  
4555 Riverside Drive  
Palm Beach Gardens, FL 33410  
Phone: +1-561-776-6700  
Fax: +1-561-776-1272

Unless otherwise indicated, as referenced herein, all trademarks are the property of Zimmer Biomet; and all products are manufactured by one or more of the dental subsidiaries of Zimmer Biomet Holdings, Inc. and marketed and distributed by Zimmer Biomet Dental and its authorized marketing partners. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. Distribution to any other recipient is prohibited. This material may not be copied or reprinted without the express written consent of Zimmer Biomet Dental. ZBINST1182 REV A 04/20 ©2020 Zimmer Biomet. All rights reserved.

